

User Instructions



CAUTION

Before use, thoroughly read MAXAIR CAPR P/N 03521015 User Instruction (received with all CAPR Helmets and available at www.maxair-systems.com>SUPPORT>USER INSTRUCTION MANUALS.



O.N. 2367-02

Quick Cuff assembled to 2081-03 Helmet

Symbol Definitions



Warning, Caution, or Note

O.N.

Order Number

PN

Part Number



Failure to follow User Instructions P/N 03521015 and the instructions contained herein may be hazardous to the user's health.

Use only if package is received unopened and contents are undamaged. If damage is noted, contact the shipper for replacement or repair.

Prior to using any MAXAIR® System or component, be sure to be familiar with the system's NIOSH approved configuration.

DO NOT use if any component is damaged. If any components are damaged or contaminated and therefore unfit for safe and effective use, they should be replaced immediately.

Only trained and experienced personnel who have read and understand the User Instructions should use MAXAIR Products.

The institution using this product in any application is responsible for determining the appropriateness of this equipment relative to regulatory requirements. Bio-Medical Devices Intl, Inc. does not recommend the appropriate systems for a particular institution or facility.

Use only MAXAIR Systems/ NIOSH approved compatible components.

NOT for use in atmospheres immediately dangerous to life or health (IDLH), and atmospheres containing less than 19.5% oxygen, or more than 25% oxygen.

Flammability Level I: fabric may burn if exposed to open flame.

Follow current local regulations governing biohazard waste to safely dispose of single use MAXAIR Products.

If you need more information, contact your BMDI Sales Representative, or call BMDI customer service at 1-800-443-3842.

For Use With

The MAXAIR 2367-02 Quick Cuff is a component of the MAXAIR "QC" Style Hood Systems utilizing the 2081-03 Helmet with a 2099-09 ChinBar attached.

MAXAIR 2367-02 Quick Cuff





Front View

Inside View

Materials

Cuff is polypropylene/polypropylene spunbonded & latex free elastic

Specifications

MAXAIR Recommended System Temperature Limits:
Use/Handling: 0°C to 54°C at a maximum 80% Relative Humidity.
Charging: 0°C to 45°C at a maximum 80% Relative Humidity.
Storage: 0°C to 35°C at a maximum 80% Relative Humidity.

Regulatory

NIOSH







The Quick Cuff is used with the MAXAIR 2081-03 Helmet.

Assembly:



1. Remove Quick Cuff from packaging and lay on the counter with label facing up.



2. Ensure proper orientation of Snaps by holding Quick Cuff up in front of ChinBar with Helmet front and Quick Cuff label facing you.



3. With Helmet turned upside down, snap one side of Quick Cuff to an appropriate Helmet Headband Side Snap Hole.



4. Snap other side of Quick Cuff to a same positioned Helmet Headband Side Snap Hole on the other side of Helmet.



5. Gently pull top of one side of Quick Cuff up and over Velcro on ChinBar.



CAUTION

Snapping to holes towards Helmet front will loosen cuff-to-chin tension; snapping to holes towards Helmet rear will tighten cuff-to-chin tension. Ensure that cuff-to-chin tension is firm and comfortable.



6. Gently pull top of other side of Quick Cuff up and over Velcro on ChinBar on other side. Pull both sides of Quick Cuff up as high on Velcro as possible.



7. With bottom of ChinBar pointed upward, pull 8. Finished assembly should bottom of Quick Cuff around ChinBar T-Tab.



appear as shown.

Disassembly:

To disassemble Quick Cuff from the Helmet, reverse steps 2-7 and discard per instutitional protocol for contaminated waste.